


Instructions for changing Filter Set

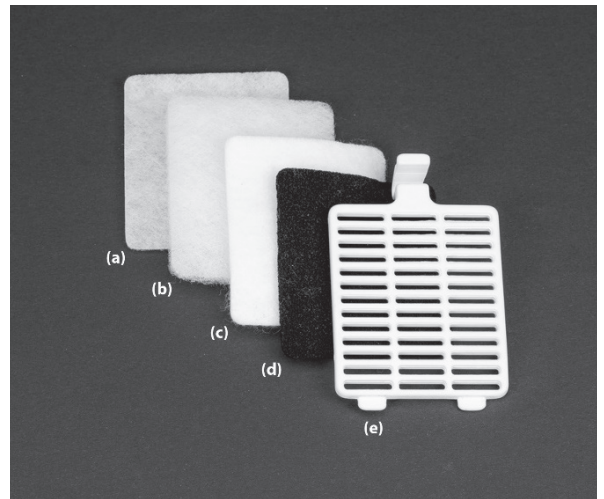

- 1) Turn off the device using the main switch and disconnect it from the power supply.
- 2) The filter grille (located at the rear of the device) can be removed by pressing the clip down and lifting the filter grille out of the device.
- 3) Remove the filter grille and all filters from the device and clean the filter cavity and the removed filter grille using a disinfectant wipe (such as Clinell). Dispose of the used filters in a zip-lock bag with your general waste.
- 4) Now, insert new filters in the following order starting from the device interior):
Sound-insulating fleece (a)
Coarse filter fleece (b)
Ultra-fine filter fleece (c)
Carbon filter foam (d)
Filter grille (e)
- 5) Then, place the filter grille back into the device.