

Rebound Achilles Wedge Kit



Optimising treatment of Achilles ruptures

The Achilles tendon is highly vulnerable to injury because of its limited blood supply and the combination of forces to which it is subjected. Current protocols suggest that immobilization and gradual reduction of plantar flexion are recommended in the treatment of Achilles ruptures.

Rebound Achilles Wedge Kit

The Rebound Achilles Wedge Kit, used in combination with the Rebound® Air Walker, provides an optimal solution for both functional (non-surgical) and post-surgical treatment of Achilles ruptures. The Rebound Air Walker provides immobilisation, protection and comfort, while the Rebound Achilles Wedge Kit provides adjustable angles of plantar flexion that can be easily modified through the rehabilitation process.

INDICATION:

- Full or partial Achilles tendon ruptures.
 - Supports functional rehabilitation:
 - During non-surgical treatment
 - After surgical treatment
- May also be beneficial for patients with Achilles tendon problems that require immobilisation in plantar flexion of the foot.

ACHILL-KIT Universal Size

Log-in to www.briggatemedical.com for pricing



SUGGESTED PROTOCOL FOR TREATMENT OF ACHILLES TENDON RUPTURES:

Suggested protocol¹ for both non-surgical and post-surgical rehabilitation for patients who have developed an acute Achilles rupture in the mid-substance of the tendon:

Week 0	Week 2	Week 4	Week 6	Week 7	Week 8
Non weight bearing circular lower leg cast with the foot at an angle of approximately 30 degrees	Rebound Air Walker with 3 Heel Wedges	Rebound Air Walker with 2 Heel Wedges	Rebound Air Walker with 1 Heel Wedge	Rebound Air Walker	Shoe with heel lift and physical therapy treatment
					

briggatemedicalcompany

Supporting Podiatrists with integrity and trust since 1987

(03) 8586 7800 (Melb. Metro)

1800 33 4142 (Free Call)

briggate@briggate.com.au

www.briggatemedical.com