

# DETERGENT WIPES

**Clinell Detergent Wipes can be used for multi-surface cleaning and damp dusting.**

To clean non-infectious surfaces and equipment.  
Not to be used on invasive equipment.

**For further information contact the Infection Prevention and Control Team.**

**The following surfaces should be routinely cleaned with these wipes\*:**

(Examples of use - this is not an exhaustive list)




**Clinell Detergent Wipes  
(Pack of 215 - CDW215AUS)**


## SURFACES & EQUIPMENT

- Chairs
- Desks
- Door handles
- Drip stands
- Worktops
- Trolleys
- Keyboards & accessories
- Lockers
- ANTT (blue tray)
- Blood pressure cuffs/machines
- Incubators
- Hoists
- Pulse oximeter probes
- Wheelchairs
- Walking frames


## INSTRUCTIONS FOR USE:

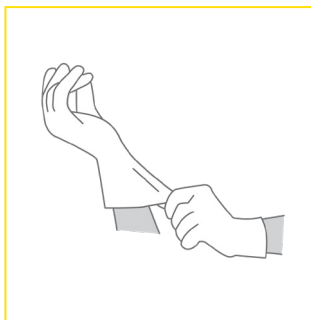


Wear recommended PPE.

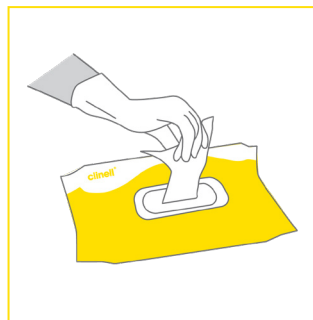

Remove one wipe from the pack. Apply to surface.

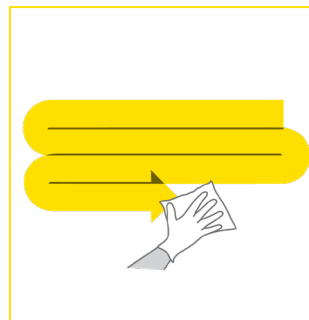

Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern, taking care not to go over the same area twice.

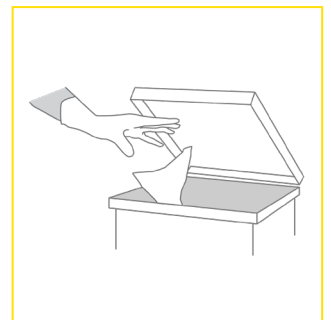

Change wipe if it becomes dry or soiled and discard. Let the surface air dry.

**DO NOT FLUSH DOWN TOILETS! DO NOT DISPOSE OF IN MACERATORS!**

\* Please ensure that you follow the manufacturer's guidelines.