

DentaLife

Clinicare Alcohol Free

Instrument Grade - Low Level Disinfectant

Composition	Benzethonium Chloride 0.40% w/v, Cetrimide 0.45% w/v, Polyaminopropyl biguanide 0.1% w/v			
Wipe Composition	Clinicare Alcohol Free wipes are available in 180 Towelettes in canister and refill pouch – Meltblown 38 gsm (15 x 17 cm) 220 Towelettes in Flat pack packaging – Meltblown 30 gsm (18.5 x 26 cm)			
Uses	Clinicare Alcohol Free Disinfectant Wipes are ideal for use on surfaces and non-critical medical devices. It has a wide spectrum antimicrobial activity and has been formulated and tested for use in healthcare and critical environments.			
Advantages	<ul style="list-style-type: none"> • Alcohol Free • Broad Spectrum, fast acting • 2 in 1 detergent and disinfectant wipe (Universal wipe) • Formulated and tested for use in healthcare and critical environments • Dermatologically Tested 			
Active Against TESTED UNDER DIRTY CONDITIONS	<i>Pseudomonas aeruginosa</i> <i>Escherichia coli</i> <i>Staphylococcus aureus</i> <i>Methicillin Resistant Staphylococcus aureus (MRSA)</i> <i>Enterococcus hirae</i> <i>Proteus vulgaris</i> <i>Klebsiella pneumoniae</i> <i>Klebsiella pneumoniae (CRE)</i> <i>Carbapenem-resistant Enterobacter cloacae (CRE)</i> <i>Acinetobacter baumannii</i> <i>Vancomycin Resistant Enterococcus faecium (VRE)</i>	(1 min) (1 min) (1 min) (1 min) (1 min) (1 min) (1 min) (1 min) (1 min) (1 min) (1 min)	<i>Salmonella enterica</i> <i>Candida auris</i> <i>Candida albicans</i> <i>Herpes simplex virus 1</i> <i>Norovirus</i> <i>Adenovirus</i> <i>SARS-CoV-2 (COVID-19)</i> <i>Vaccinia virus (mpox)</i> <i>Influenza virus (H1N1, H3N2)</i> <i>Respiratory syncytial virus (RSV)</i>	(1 min) (1 min) (1 min) (1 min) (1 min) (1 min) (30 sec) (1 min) (1 min) (1 min)
Contraindications	Not to be mixed with any detergents, chemicals or other cleaning agents.			
Clinical Procedure	When decontaminating and disinfecting surfaces thoroughly wipe the surface in an S-shape pattern, from high to low, from clean to dirty. Leave the surface to remain wet for 1 minute. Allow to air dry.			
Poison Schedule	Not Applicable			
Safety Directions	Avoid contact with eyes. Wash hands after use. DO NOT use as a personal or baby wipe. Avoid contact with damaged skin. Wear recommended PPE			
First Aid	For advice, contact a Poisons Information Centre. Australia 13 11 26, New Zealand, 0800 764 766			
Regulatory	Class IIbARTG: 405881			
Presentations	DL2321 Clinicare AF Instrument Grade - Low Level Disinfectant Wipes Canister (180) DL2321R Clinicare AF Instrument Grade - Low Level Disinfectant Wipes Refill Pouch (180) DL2371 Clinicare AF Instrument Grade - Low Level Disinfectant Flat Pack (200 Wipes) DL2378 Clinicare AF Instrument Grade - Low Level Disinfectant Flat Pack (30 Wipes)			
Storage	Store below 30 °C in a cool, dry place.			
Shelf Life	2 years from date of manufacture as indicated on pack for Canisters and Flatpacks. 3 years from date of manufacture as indicated on pack for Refills.			

