



SÜDA Replacement Vacuum Motor Filter

The Vacuum Motor Filter needs to be changed after 5 Dust Bag changes.
Replace the Filter as detailed below:


1. Remove the **Dust Bag Faceplate**
(this is where the Hose enters the Main Control Unit).

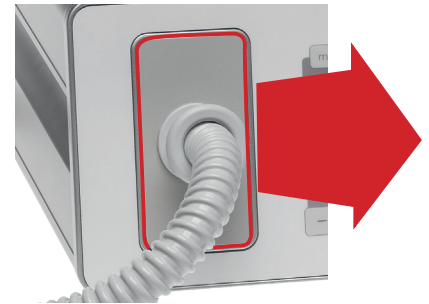

2. Remove the existing **Vacuum Motor Filter** from within the Dust Bag Chamber (located on the rear right-hand wall as pictured).

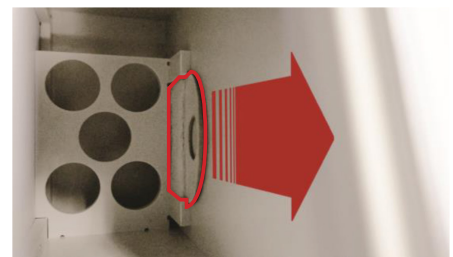

3. Push the new Filter into place. Ensure that both the top and bottom edges of the Filter are positioned behind the **guide rails**.

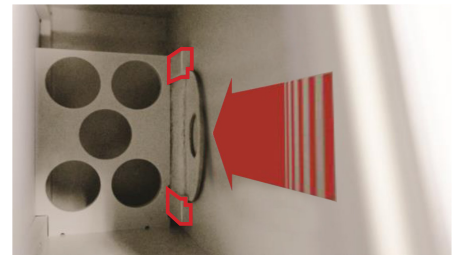

4. Take the Dust Bag Faceplate with a fitted Dust Bag and gently push it into the Dust Bag Chamber. Ensure the Handpiece power pins (located at the top and bottom behind the Dust Bag Faceplate) are properly aligned to the Main Control Unit.

5. Ensure the Dust Bag Faceplate is connected properly to the Main Control Unit prior to operation.

briggatemedicalcompany

Supporting Podiatrists with integrity and trust since 1987