

Gebrauchsanleitung

Instructions for use

Hallux Valgus Nachtschiene

Hallux Valgus Night Splint

dynamics®

ofa bamberg

Die vielseitige
orthopädische Versorgung
Versatile orthopaedic care

1



2



 **Important information**

- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.






Important information

Hallux Valgus Night Splint

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- The splint can be cleaned using a damp cloth.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Hand wash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

- Hallux Valgus (bunion)
- Formation of bony lumps
- Deformed big toe
- For stabilisation after bunion surgery

Material composition

- 38% Acrylonitrile butadiene styrene
- 22% Polyurethane (foam)
- 20% Polyamide microfibre
- 18% Polyamide
- 2% Polyurethane

Side effects

There are currently no known side effects for any part of the body if the product is used correctly. However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

Contraindications

- Consult your doctor before using this product in the following cases:
- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
 - Feeling of numbness and circulatory problems in the area where the product is used
 - Problems with lymphatic drainage and unexplained swelling

Disposal

Please dispose of the product according to local regulations after the end of use.

dynamics®

Wir bringen Sie schnell wieder in Bewegung.

We'll get you moving again in no time.

Nous vous remettons en mouvement
en un rien de temps.

Wij brengen u snel weer in beweging.

Vi rimettiamo velocemente in movimento.

Le ayudamos a recuperar la movilidad rápidamente.

Em breve estará em movimento.

Rychle vás znovu rozpohybujeme.

Vi får dig snabbt i rörelse igen.

Avullamme pääset taas nopeasti liikkeelle

Σύντομα θα κινείστε και πάλι.

אנחנו נחזיר לך את הניידות במהירות.





aus 100 %
recycltem Papier



ofa bamberg

Ofa Bamberg GmbH

Laubanger 20

D-96052 Bamberg 

Tel. + 49 951 6047-0

Fax + 49 951 6047-185

info@ofa.de

www.ofa.de

ofa austria

Ofa Austria

Wasserfeldstr. 20

A-5020 Salzburg

Tel. + 43 662 848707

Fax + 43 662 849514

info@ofaaustria.at

www.ofaaustria.at