

**Removing**

<p>Hold the instrument changing aid horizontally to the handpiece.</p>	
--	--

<p>Position the instrument head behind the notch on the instrument changing aid. Place your thumb over the notch on the instrument changing head so that the instrument does not drop when removing it.</p>	
---	--

<p>Carefully pull the instrument from the clamping the device of the handpiece using the instrument changing aid.</p>	
---	--

ATTENTION! Please observe!

ATTENTION! Please observe!

Dear customer!

We are delighted to be able to inform you in this information sheet that our newly developed, innovative and patented Easy-Clean clamping device (referred to as "Easy-Clean" in the following) is fitted in your handpiece.

In a conventional clamping device, the dirt that is produced while working may mean that the instrument that you are operating with your handpiece may stop even though the motor is in operation. This can be accelerated by creams, oils, grease or the like.

The result: The clamping device is no longer functional or only has limited functionality and until now had to be sent to the factory for maintenance and cleaning.

We have developed Easy-Clean to save you having to do this in future. You can clean Easy-Clean yourself very easily and without taking much time and energy.

We recommend that you clean Easy-Clean every 14 days (even earlier if necessary).

The following devices are marked for the installation of Easy-Clean:  
**A500, A700, A800, A900, A950, A1000, A1100, A2000, S900, S950, S1000, Euro-Spray, Compact, Rondo**  
**All other devices are NOT marked for the installation of Easy-Clean.**

<p>Circle = marking for EasyClean clamping device</p>
<p>The marking on the <b>Euro-Spray</b> is not applied on the handpiece tip. You will find the marking on the Euro-Spray by removing the handpiece tip, where a spot is milled into the clamping unit.</p>

ATTENTION! Please observe!

**Safety instructions**

- It is essential that you read the following cleaning instructions with the utmost care! They contain important information on protecting yourself, others and your device from damage! The device must only be used by trained professionals.
- Please observe the operating instructions for your device!
- Never perform cleaning work on a device that is still connected to the mains. As described in the operating instructions for your device, please switch off the device beforehand at the main switch and remove the safety plug from the socket.
- Caution: risk of injury! Never attempt to perform cleaning work on the handpiece while the handpiece motor is running. This must only be done when the handpiece motor is switched off.**
- Only the tool supplied with this information sheet may be used to clean Easy-Clean. Other types and fields of application are carried out at your own risk, and may conceal dangers. No form of misappropriation is permitted. Improper use may lead to damage to persons or objects. The manufacturer cannot be held responsible for damages caused by improper use, unqualified personnel or incorrect operation. **In the event of improper use, any guarantee claims shall be invalidated.**
- The clamping device cleaners must only be used for Easy-Clean (developed for self-cleaning). **Under no circumstances** must the clamping device cleaners be used for conventional clamping devices that are not intended for self-cleaning, as they may cause damage to the clamping device.
- Clean clamping device cleaner 1 after every use and make sure that it is always in an immaculate hygienic condition before use. We recommend a brush (Art. No. 34916) for cleaning clamping device cleaner 1.
- For reasons of hygiene, please clean clamping device cleaner 2 under running water and then with alcohol after every use.
- The figures are for illustration purposes only. They may be subject to technical and optical changes.

To clean Easy-Clean you will find the following tools/aids provided with this information sheet:

<p>1x Clamping device cleaner 1 Universal stainless steel <b>Art No. 40287</b></p>	
<p>1x Clamping device cleaner 2 (Cleaning brush for Easy-Clean) <b>Art No. 40269</b></p>	

To clean Easy-Clean, please proceed as follows:

1. As described in the operating instructions for your device, please switch off the device at the main switch and remove the safety plug from the socket. Never clean the device while the handpiece motor is still running! **Risk of injury! The clamping device will also be damaged!**
2. Put on a disposable glove or put a finger cot on the thumb of the hand that you wish to use to hold the handpiece during cleaning, then hold the handpiece in this hand.
3. Now push on the tip of the handpiece with your thumb. By holding the tip with the disposable glove/finger cot, you will prevent the tip from turning as well during cleaning (see Fig. 1).
4. Without applying pressure, insert the clamping device cleaner 1 into the opening of your handpiece as far as possible and turn the clamping device cleaner 1 left and right with gentle pressure (see Fig. 2).
5. Now remove the clamping device cleaner 1 from the opening. Next, use a small brush to remove the dirt from the grooves of the clamping device cleaner 1. Repeat this cleaning process until you cannot remove any more dirt.
6. After use, clean the clamping device cleaner 1 with the Baehr quick surface disinfectant (Art. No. 11015).
7. Now remove the last impurities with the clamping device cleaner 2 by inserting it into the opening of your handpiece as far as possible and turning it left and right (see Fig. 3).
8. Now remove the clamping device cleaner 2 from the opening.
9. Clean the clamping device cleaner 2 with just water first and then using the Baehr quick surface disinfectant (Art. No. 11015) followed by Baehr alcohol (Art. No. 11032) to degrease the tool.

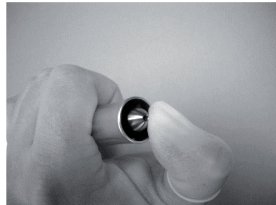


Fig. 1

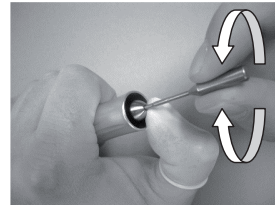


Fig. 2

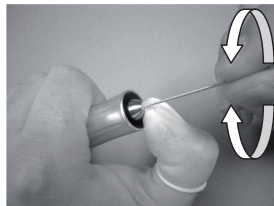


Fig. 3

If cream, fat, oil or the like ever enter the clamping device, you can also use Easy-Clean for cleaning in this case.

To do this, please proceed as follows:

1. As described in the operating instructions for your device, please switch off the device at the main switch and remove the safety plug from the socket. Never clean the device while the handpiece motor is still running! **Risk of injury! The clamping device will also be damaged!**
2. Moisten the clamping device cleaner 2 in alcohol (Art. No. 11032) (**never** use greasy disinfectant or other oily substances, as Easy-Clean may no longer function correctly otherwise).
3. Insert the moistened clamping device cleaner 2 into the opening of your handpiece and turn it left and right.
4. Remove the clamping device cleaner 2.
5. Now leave the handpiece to air dry for at least 3 minutes.

Your handpiece will now be ready for operation once again.

If your handpiece still does not function correctly in spite of these cleaning measures, or if you have and questions on the cleaning process, please get in touch with us.

The Gustav Baehr GmbH team hopes that you enjoy your successful work with your new Easy-Clean clamping system.



Gustav Baehr GmbH  
 Max-Eyth-Str. 39 - D-71332 Waiblingen  
 Tel.: 07151-95 90 20 - Fax: 07151-18 444  
 www.baehrshop.de

Instrument changing aid (Insertion and removal assistance for very small instruments)

Dear customer!

To accompany your Easy-Clean clamping device, we have developed an instrument changing aid which is intended to make it easier for you to change (remove and insert) very small instruments (instrument diameter: approx. 1.5 mm – approx. 5.0 mm).

Safety instructions

	It is essential that you read the following instructions with the utmost care! They contain important information on protecting yourself, others and your device from damage! The device must only be used by trained professionals.
	Please observe the operating instructions for your device!
	Never perform an instrument change on a device that is still connected to the mains. As described in the operating instructions for your device, please switch off the device beforehand at the main switch and remove the safety plug from the socket.
	<b>Caution: risk of injury! Never attempt to perform an instrument change on the handpiece while the handpiece motor is running. This must only be done when the handpiece motor is switched off.</b>
	Only the tool supplied with this information sheet may be used to change instruments. Other types and fields of application are carried out at your own risk, and may conceal dangers. No form of misappropriation is permitted. Improper use may lead to damage to persons or objects. The manufacturer cannot be held responsible for damages caused by improper use, unqualified personnel or incorrect operation.
	<b>In the event of improper use, any guarantee claims shall be invalidated.</b>
	Clean the instrument changing aid after every use and make sure that it is always in an immaculate hygienic condition before use.
	For reasons of hygiene, please clean the instrument changing aid under running water and then with Baehr quick surface disinfectant (Art. No. 11015) after every use.
	The figures are for illustration purposes only. They may be subject to technical and optical changes.

Inserting

Insert the instrument shaft into the clamping device of the handpiece.

Hold the instrument changing aid horizontally to the handpiece.

With the countersink on the head of the instrument changing aid, insert the instrument into the handpiece as far as the stop.

ATTENTION! Please observe!

ATTENTION! Please observe!

ATTENTION! Please observe!

ATTENTION! Please observe!