JOBST® Compri2

JOBST® Compri2 lite

A

How to Apply

www.jobst.com/C2-no2

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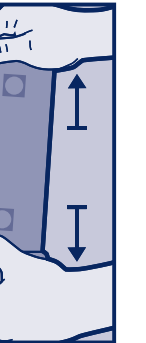
EN The outer compression bandage
It is recommended to keep the bandage close to the foot and lower angle during application to obtain even compression.

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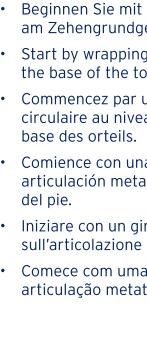
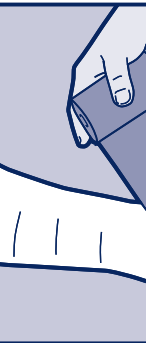
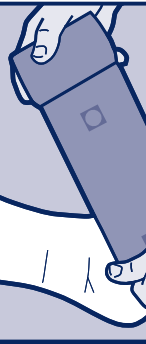
• An indicator on the bandage helps to ensure correct tension by changing from an oval to a circle when it is stretched.

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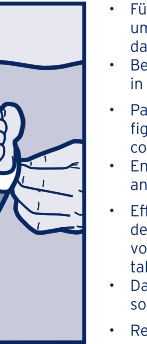
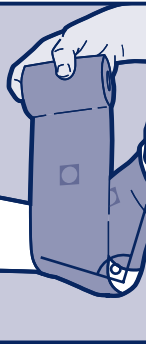
• Start by wrapping it around the foot at the base of the toes.

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• Pass it around the ankle in two or three figures-of eight, ensuring that the heel is completely covered.

• Ensure that the foot is kept at a right angle.

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• Continue wrapping the bandage around the lower leg, overlapping it at least 50 %, keeping the bandage under even tension. The indicator on the bandage helps to achieve the desired pressure profile.

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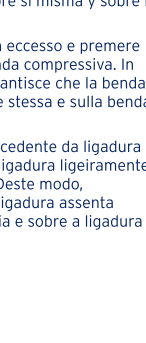
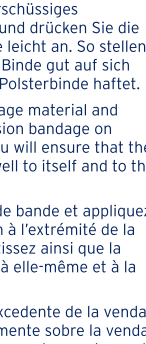
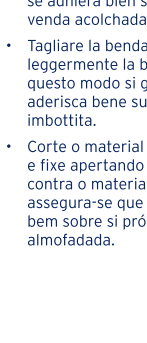
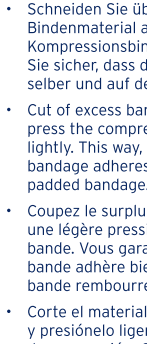
• Cut off excess bandage material and press the compression bandage on lightly. This way, you will ensure that the bandage adheres well to itself and to the padded bandage.

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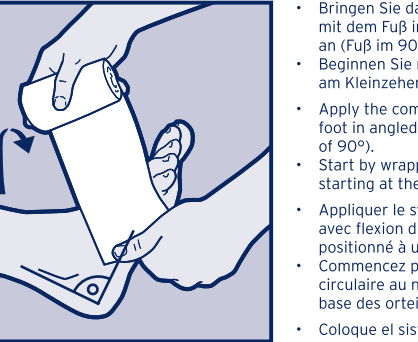
EN Padded bandage with light compression
Apply the padded bandage with the shiny side next to the skin. Stretch the bandage only enough for it to adjust to the shape of the leg. The overlapping rounds of the bandage should ensure adequate padding on the one hand; on the other hand a thin compression dressing makes it easier to wear shoes.

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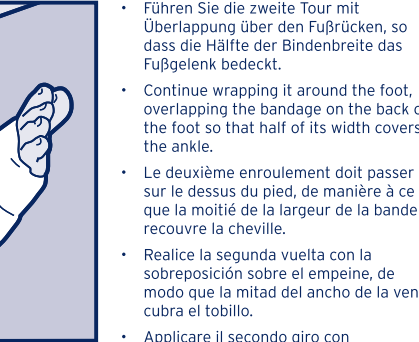
- Apply the compression system with the foot in angled position (foot at an angle of 90°).
- Start by wrapping it around the foot starting at the base of the toes.

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
- Continue wrapping it around the foot, overlapping the bandage on the back of the foot so that half of its width covers the ankle.

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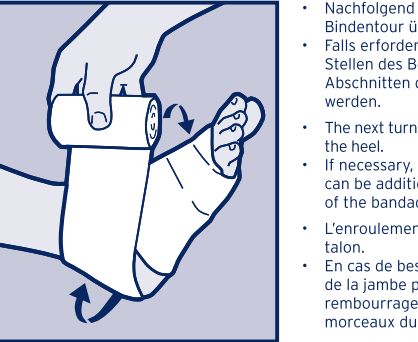- The next turn of the bandage goes over the heel.
- If necessary, prominent points on the leg can be additionally padded with off-cuts of the bandage.

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BSN medical GmbH