



competency in hygiene

# 23S+ or 23VS+

## what is the difference?

Euroklav® 23S+



Euroklav® 23VS+



The difference is technical, however in summary the VS model (V representing "vacuum") utilises it's vacuum motor to place the chamber into a slight negative pressure throughout the sterilisation and drying cycle. By doing this, it allows the steam to penetrate throughout the instrument load with a little bit more aggression whilst also enhancing the drying cycle's effectiveness.

In doing this the VS model is "mimicking" what a full-on B-Class autoclave does. (A B-Class autoclave places the chamber into a heavy negative pressure that allows the steam to penetrate into HOLLOW instruments. For example; instruments that a dentist may use).

What does this mean to you? It may allow you to load the chamber with an extra one or two packs and it also cuts the drying cycle time down by 5-7 minutes.

The standard S model does not utilise its vacuum motor to the same effect.

Both the S and the VS are both classified as S-Class autoclaves. S-Class autoclaves are suitable for Podiatry instruments which are classified as SOLID (i.e: NOT hollow).

# briggatemedicalcompany

Supporting Podiatrists with integrity and trust since 1987

(03) 8586 7800 (Melb. Metro)

1800 33 4142 (Free Call)

briggate@briggate.com.au

www.briggatemedical.com