




Cleansing sodium chloride dressing

Mesalt effectively stimulates the cleansing of heavily discharging and discharging infected wounds.

Absorbent viscose/
polyester nonwoven
• Impregnated with
sodium chloride.

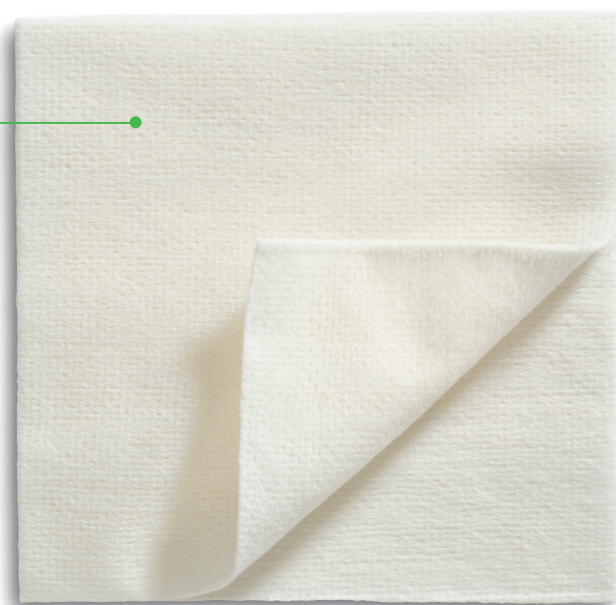

How Mesalt® works


The discharge from the wound releases the sodium chloride from the dressing. Mesalt effectively stimulates the cleansing of wounds in the inflammatory phase by absorbing exudate, bacteria and necrotic material from the wound, thereby facilitating the natural wound healing process.

Areas of use


Mesalt is intended for the management of heavily discharging and discharging infected wounds in the inflammatory phase such as:

- Pressure ulcers
- Surgical wounds


How to use Mesalt

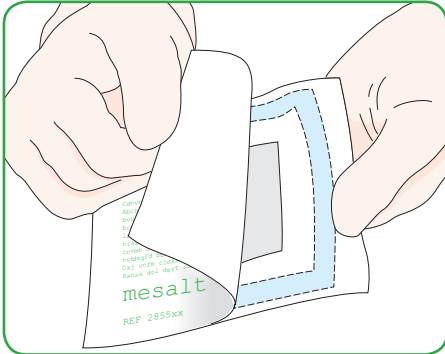

Clean the wound and open the package.

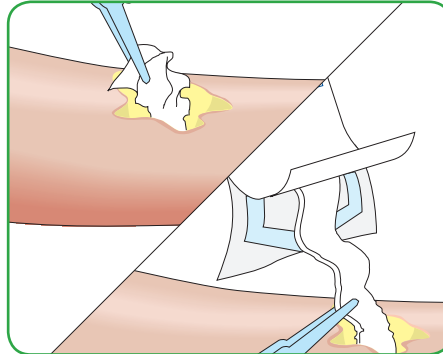

Apply or loosely pack Mesalt (dry) to the wound. Do not soak before-hand.

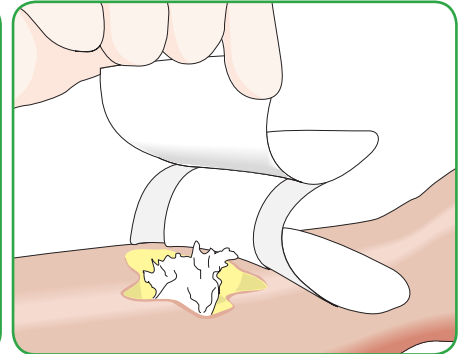

Apply Mepilex Border as cover dressing. Depending on the amount of exudate, Mesalt is normally changed daily, or more often if necessary.

Benefits of Mesalt

- Stimulates debridement
- Reduces bacterial burden
- Easy to use

Note

Mesalt is not to be used in dry wounds. The use of Mesalt has been presented in papers and study reports. The data is available from Mölnlycke Health Care on request.



Mesalt is packaged sterile in single packs

Art. No	Size cm	Pcs/shelf cont.	Pcs/transp. cont.
Mesalt® 285580	5 x 5	30	240