Made in England by

OWEN MUMFORD

www.owenmumford.com

UK / EU
Owen Mumford Ltd, Brook Hill
Woodstock, Oxford OX20 1TU England
T : 01993 812021
F : 01993 813466
E : info@owenmumford.net

NORTH AMERICA
Owen Mumford Inc
1755-A West Oak Commons Court
Marietta, Georgia 30062, USA
T : 800-421-6936
F : 770-977-2866
E : info@owenmumfordinc.com

FRANCE
Owen Mumford, BP 444
27204 Vernon Cedex, France
T : 02 32 51 88 70
F : 02 32 21 96 95
E : contact@owenmumford-fr.com

GERMANY
Owen Mumford GmbH
Sternplatz 3, D-97896 Freudenberg/M,
Germany
T : 09375/92990
F : 09375/929929
E : info@owen-mumford.de

EXPORT
Owen Mumford Ltd, Brook Hill
Woodstock, Oxford OX20 1TU United
Kingdom
T : +44 1993 812021
F : +44 1993 813473
E : export@owenmumford.co.uk

Neuropen® is a registered trademark of Owen Mumford.

Neurotips™ is a trademark of Owen Mumford.

0120

NT/0100/11/0000/02

Neuropen®
NT0100

Neurotips™
NT5405 (box of 100)
NT5405 (boîte de 100)
PZN 0708383 (Packung mit 100 Stück)
NT5405 (caja de 100)
NT5405 (Scatola da 100)
NT5405 (ask med 100)

Monofilaments

10g Monofilaments (white):
NT0104 (box of 5).
Monofilament (blanc): pression 10g:
NT0104 (boîte de 5).
Bestellnr. für Monofilamente von 10g (weiß):
PZN 0708360 (Packung mit 5 Stück).
Monofilamento de repuesto de 10g (blanco)
Código: NT0104 (caja de 5).
10g Monofilamento (Bianco):
NT0104 (Scatola da 5).
10g Monofilament (vit):
NT0104 (ask med 5).

ENG *Neuropen® has been designed as a consistently reliable, dual purpose device for testing large nerve fibres in the foot for peripheral neuropathy. Its beneficial features include:*

- an accurately calibrated inter-changeable 10g (white) monofilament for touch/pressure assessment.*
- sterile 'Neurotip' for assessing pain.*
- convenient pocket size.*
- hard casing, which protects the monofilament when not in use, with finger grips and clip.*

The monofilament is calibrated to bend at 10g of force.

COMPONENTS

- A. Neurotip inserted in the calibrated Neurotip carrier
- B. Neurotip pressure gauge
- C. 40g indicator
- D. Pocket clip
- E. Button
- F. Slot
- G. Sliding monofilament
- H. Barrel

General care and replacements

Cleaning

The monofilament and the outside of the device may be cleaned using an alcohol wipe or antiseptic solution. **IMPORTANT: never immerse your Neuropen in liquid or attempt to re-sterilise a Neurotip.**

Replacement Neurotips

Extra Neurotips can be obtained from the supplier of this Neuropen. In case of difficulty please contact your local Owen Mumford office. Order code: NT5405 (box of 100). Remove and discard the used Neurotip in an appropriate sharps container.

Replacing the Monofilament

Monofilaments lose their accuracy if used regularly, and should be replaced after 100 uses.

Soiled or worn monofilaments should be replaced by following the instructions below.

- With the monofilament button positioned halfway along the slot in the barrel, place thumb on button and slide back towards pocket clip and click into place. The monofilament is now in the locked position.
- To release the monofilament from the casing, from the locked position, click firmly back again. The monofilament can now be removed by holding the button and pulling away from the Neuropen. Discard in a sharps container.
- Ensure that the new monofilament is correctly inserted into the side of the device, which is the same side as the 40g indicator (white square) and pressure gauge.
- Take the new monofilament button between thumb and finger and feed through slot so that it appears through the hole in the end of the device.
- Position the button halfway along the slot and click firmly into place.

CAUTION: If monofilament is used near an open sore, replace immediately with a new monofilament after use. When not in use, keep monofilament in locked position for protection.

WARRANTY INFORMATION

This product is guaranteed against manufacturing defects for two years from date of purchase. In the unlikely event of failure, please contact your local Owen Mumford office. Returning the device will not affect your statutory rights.

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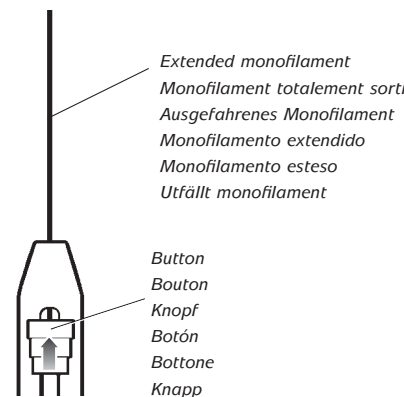
ENG Assessing protective touch/pressure sensation

Monofilament Test

1. Prepare patient according to guidelines.
2. Fully extend the monofilament by sliding the button to the end of the device until a click is heard.
3. Wipe the end of the monofilament with an alcohol wipe or antiseptic solution.
4. Press the monofilament at a 90° angle to the skin surface and increase the pressure until it bows.
5. Hold in position for 1 – 2 seconds.
6. Suggested sites to be tested are shown in the diagram and should be tested at random.

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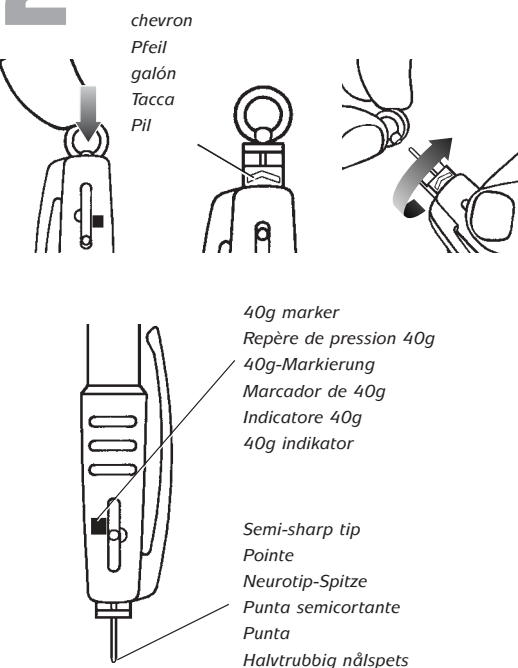
2

Assessing protective pain/sharpness sensation

Calibrated Neurotip Test

This is a calibrated and sterile pain/sharpness test. The spring mechanism is calibrated to exert a force of 40g, which will help to identify patients with a loss of pain sensation.

1. Take an **unused** Neurotip and hold by the cap.
2. Press Neurotip firmly down into the Neurotip carrier of the Neuropen as far as it will go and clicks into position. The Neurotip is correctly positioned when only **one chevron** can be seen on the Neurotip.
3. To expose the sterile semi-sharp tip, place thumb gently over the Neurotip pressure gauge and remove Neurotip cap by twisting and pulling outwards.
4. Press the Neurotip against the test surface at a 90° angle. Hold in place for 1-2 seconds before removing, taking care to **depress within the 40g marked white area**.
5. Ask the patient if they detect a sharp sensation.
6. **IMPORTANT: Remove and discard the used Neurotip in an appropriate sharps container.**

CAUTION: Do not use if Neurotip cap has previously been removed.

There is no difference in the force reading between a red and white Neurotip when used within the Neuropen. They do however provide a contrast when used for peripheral vision testing.

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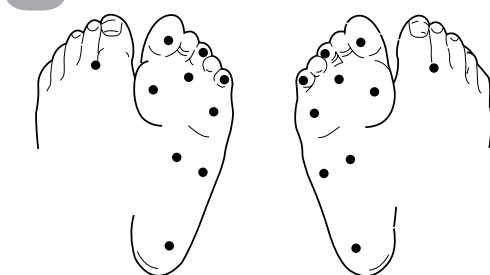
3

Comparative sensory testing

Neuropen makes it possible to easily assess whether the patient can distinguish between a touch stimulus (monofilament) and a sharp stimulus (Neurotip).

Randomly test the patient with either the Neurotip or monofilament by inverting the pen and ask the patient to identify which stimulus they feel.

A normal individual will easily discriminate between these sensations – an inability to distinguish identifies patients at risk of peripheral neuropathy.

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