

CORD REPLACEMENT KIT



(1) Allen wrench(1) Brace Plate(4) Screws

www.saebo.com/saebostep



With the included Allen wrench, loosen and remove all 4 screws from the existing Brace Plate.





Remove and discard existing Brace Plate.



Align new Brace Plate so the holes are matching with the SaeboStep Cuff.



Secure and tighten the new Brace Plate using the included Allen wrench and screws.



Saebo, Inc.

2459 Wilkinson Blvd., Suite 120-B Charlotte, NC 28208

> 888-942-5751 sales@saebo.com www.saebo.com

EC REP

European Healthcare & Device Solutions (Ireland) Ltd. info@europeandevicesolutions.co.uk www.europeandevicesolutions.co.uk

> Australian Sponsor: Ausmedic Australia Pty Ltd Unit 3, 3 Basalt Road Pemulwuy NSW 2145 1300 473 422 Performhealth com au







