



1. In the Main Menu click "Patient Information". Enter the patient's details, click "Save" and then click "Return to Main Screen".

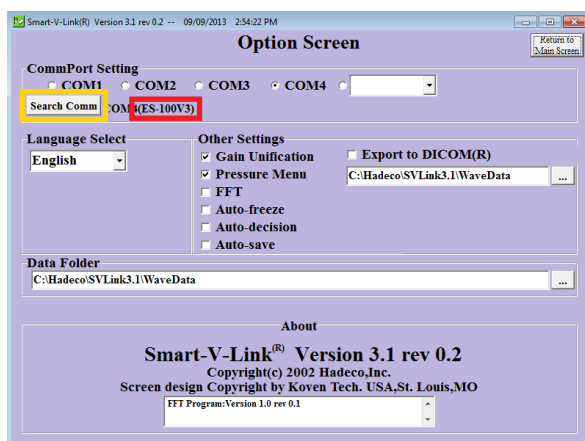


2. Plug the Doppler into your computer, switch it on and click "Options". Click "Search Comm". The program will find the Doppler and display which CommPort the unit is plugged in to. Doppler model number will also appear. Click "Return to Main Screen".

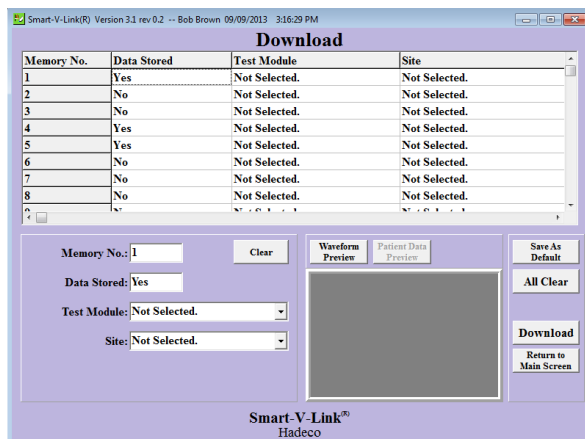


Memory No.	Data Stored	Test Module	Site
1	Yes	Not Selected.	Not Selected.
2	No	Not Selected.	Not Selected.
3	No	Not Selected.	Not Selected.
4	Yes	Not Selected.	Not Selected.
5	Yes	Not Selected.	Not Selected.
6	No	Not Selected.	Not Selected.
7	No	Not Selected.	Not Selected.
8	No	Not Selected.	Not Selected.


3. Click "Download" option from the Main Screen. A list of tests saved within the Doppler will appear.

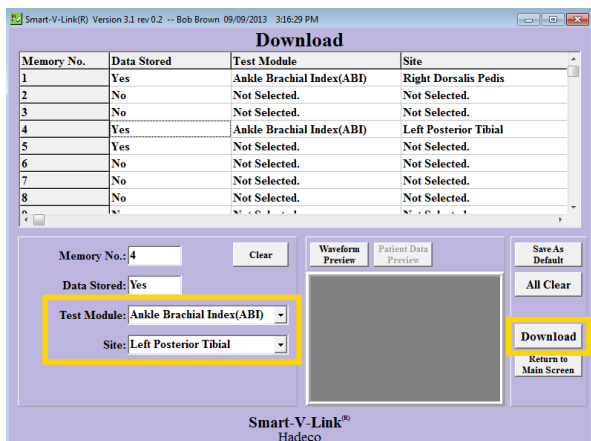


- Click on tests for this particular patient and select the Ankle Brachial Index (ABI) from the "Test Module" drop-down menu (most useful screen for your requirements). Next, select the relevant test site for each test from the "Site" drop-down menu. Once the tests have had the appropriate Site nominated to them, click "Download". Click "Return to Main Screen".

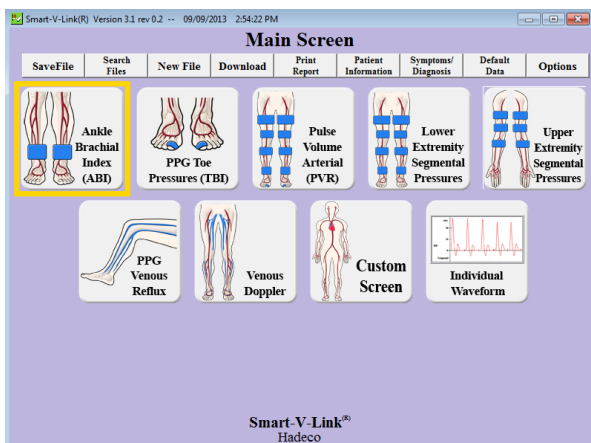


- From the Main Screen, click on "Ankle Brachial Index (ABI)".

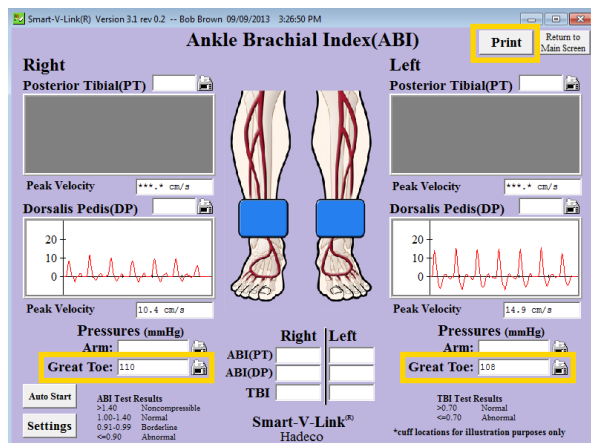

- In the ABI screen, you will find the selected patient's Waveforms. You can now manually enter the patients **Toe Pressures**. Once completed, click Print.

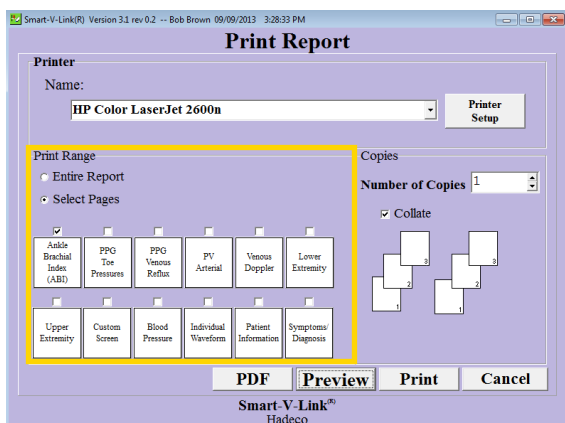

- In the "Print Range" section, use the check box to select the report you wish to print. If only using the ABI screen, that is all that is required to be ticked. Click "PDF" to export the report as a PDF file or click "Print" to print to the printer set-up on your device.

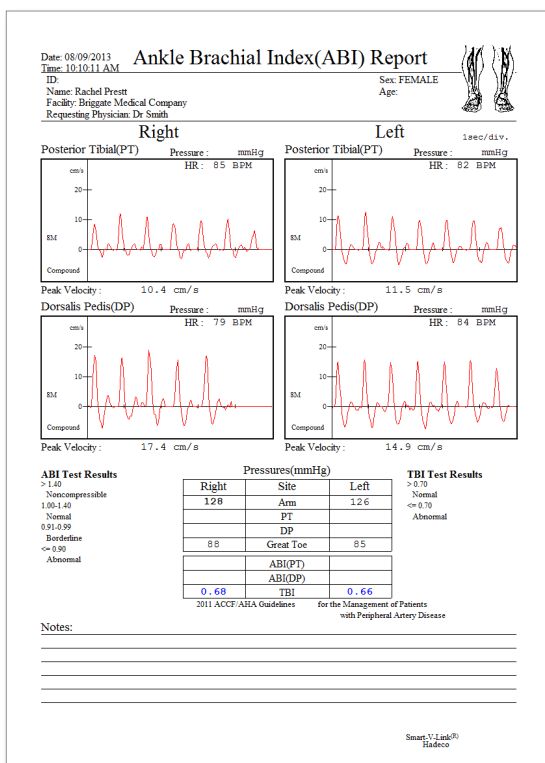

Example of an Ankle Brachial Index (ABI) screen report.