

PegContour Insole

Off-loading Cushion Insole

The effective, three-dimensional insole system for full pressure distribution as well as optional selective off-loading



- > optimal pressure redistribution through its transverse arch and medial and lateral longitudinal arch
- > targeted off-loading by removing individual pegs
- > right and left version available



DARCO

PegContour Insole

with two different sole shapes



Fits for



AllRound® Shoe

Closed long-term Healing Shoe



Fits for



Relief Dual®

Off-loading Shoe




MedSurg

Healing Shoe



MedSurg Pro®

Healing Shoe

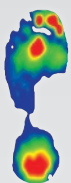


Neutral
measurement



AllRound Shoe®



PegContour Insole
(MTP II + III spared)
in the AllRound Shoe®

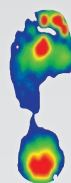

Neutral
measurement




Relief Dual®



PegContour Insole
(MTP II + III spared)
in the Relief Dual®

Removal of individual pegs enables a customised fit – postoperatively or for acute wound care.

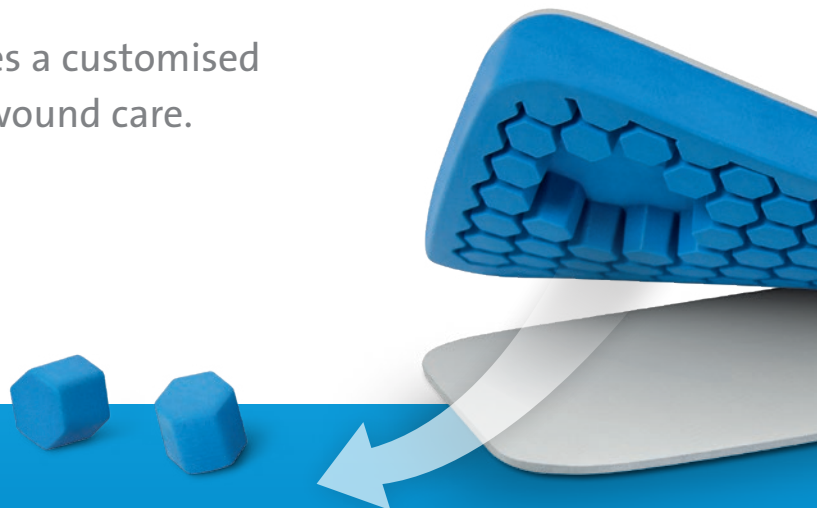

INDICATIONS

- > for use after hallux valgus and small toe surgeries (Austin, Akin, Scarf, Weil, Hohmann, etc.)
- > talipes valgus, flat foot, splayfoot
- > angioneuropathic changes of the foot in combination with mild foot deformities

Selective off-loading for:


- > plantar lesions
- > plantar ulcers /ulcer prevention
- > pressure points

PURPOSE

- > effective insole system for plantar pressure relief in acute care
- > pressure can be reduced by up to 55 %
- > optimal pressure redistribution is achieved through the contours

FEATURES

- > soft lengthwise and crosswise sole covering made of functional polyester
- > customisable PU material
- > customised fit through the removal of single pegs
- > available for the right and left foot

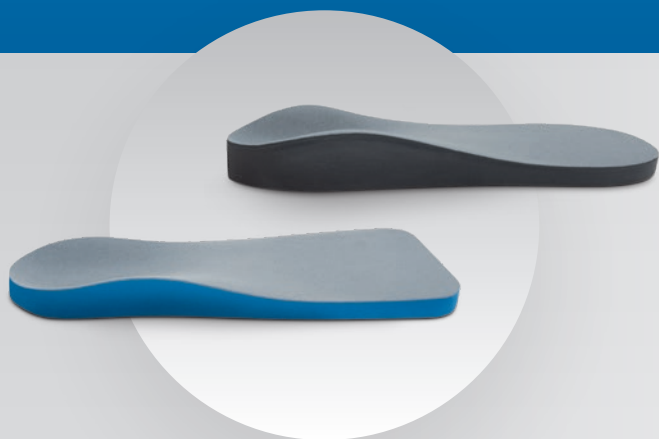

The 15 mm high PegContour Insole, made from 100% PU material, provides a high level of wearing comfort with the lengthwise and crosswise stretchable sole covering.




After the pegs have been removed from the area that needs relief (Figure 1 + 2), the sole base supplied is affixed to the entire sole surface (Figure 3). A thin layer of EVA (sole base) stabilises the off-loading area, and keeps the remaining pegs in place.

Article List

PegContour Insole for AllRound Shoe®

All measurements are given in cm and correspond to the size of the insole.

SIZE	ART. NO.	ART. NO.
	Left	Right
XS 37,0 – 38,0	PCI-ARS0-L	PCI-ARS0-R
S 39,0 – 40,0	PCI-ARS1-L	PCI-ARS1-R
M 41,0 – 42,0	PCI-ARS2-L	PCI-ARS2-R
L 43,0 – 44,0	PCI-ARS3-L	PCI-ARS3-R
XL 45,0 – 46,0	PCI-ARS4-L	PCI-ARS4-R
XXL 47,0 – 48,0	PCI-ARS5-L	PCI-ARS5-R

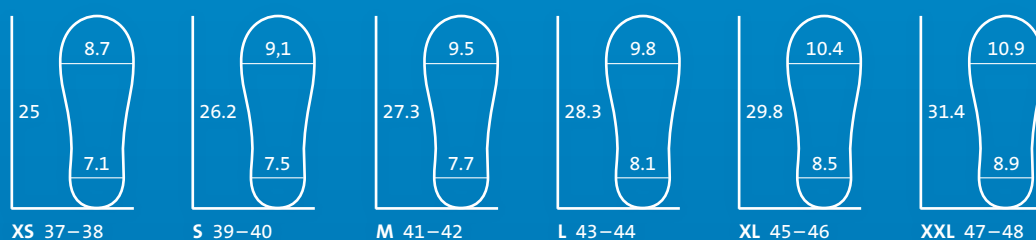

PegContour Insole for Relief Dual®, MedSurg, MedSurg Pro®

SIZE	ART. NO.	ART. NO.
	Left	Right
Women		
WS 34,0 – 36,5	PCI-W1-L	PCI-W1-R
WM 37,0 – 38,5	PCI-W2-L	PCI-W2-R
WL 39,0 – 41,0	PCI-W3-L	PCI-W3-R
Men		
MS 39,0 – 41,0	PCI-M1-L	PCI-M1-R
MM 41,5 – 43,0	PCI-M2-L	PCI-M2-R
ML 43,5 – 45,0	PCI-M3-L	PCI-M3-R
MXL 45,5 – 47,0	PCI-M4-L	PCI-M4-R

